

TESTS AND READING TIME

LEU	LEUKOCYTES 2 minutes	NEGATIVE		TRACE	SMALL +	MODERATE ++	LARGE +++	
NIT	NITRITE 60 seconds	NEGATIVE			POSITIVE (any degree of uniform pink color)			
URO	UROBILINOGEN 60 seconds	NORMAL mg/dL URINE (1 mg = approx. 1 EU)						
		0.2	1		2	4	8	
PRO	PROTEIN 60 seconds	NEGATIVE	TRACE	mg/dL	30 +	100 ++	300 +++	2000 or more ++++
pH	pH 60 seconds	5.0	6.0	6.5	7.0	7.5	8.0	8.5
BLO	BLOOD 60 seconds	NEGATIVE	NON-HEMOLYZED TRACE	MODERATE	HEMOLYZED TRACE	SMALL +	MODERATE ++	LARGE +++
SG	SPECIFIC GRAVITY 45 seconds	1.000	1.005	1.010	1.015	1.020	1.025	1.030
KET	KETONE 40 seconds	NEGATIVE	mg/dL	TRACE 5	SMALL 15	MODERATE 40	80	LARGE 160
BIL	BILIRUBIN 30 seconds	NEGATIVE			SMALL +	MODERATE ++	LARGE +++	
GLU	GLUCOSE 30 seconds	NEGATIVE	g/dL (%) mg/dL	1/10 (ir.) 100	1/4 250	1/2 500	1 1000	2 or more 2000 or more